OUTPAW

PET IDENTIFICATION

USER MANUAL

365
DAYS

Outpaw
Customer
Support

Gebrauchsanleitung	3 - 10
User Manual	11 - 18
Manuel d'instruction	19 - 26
Manual de Instrucciones	27 - 34
Manuali di istruziane	35 - 42
Instrukcja obsługi	43 - 50
Handbok	51 - 58
Manual de Instrucțiuni	59 - 66
Наръчник на потребителя	67 - 74
Руководство по експлуатации	75 - 82
Kullanım Kılavuzu	83 - 90
Konformitätserklärung	91
Declaration of Conformity	91

OUTPAW

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OUTPAW

USER MANUAL

About the manual: Please read the manual carefully. Please note and follow all information mentioned in this manual to ensure a long service life and reliable use of your device. Store this manual within reach and pass it on to other users of the device.

Content


- 1x Outpaw PetIdent 160
- 1x USB to Micro-USB charging cable
- 1x User manual
- 1x Transport case

Specifications

Model	:	Outpaw PetIdent 160
Input charging	:	5V
Connectivity	:	USB - Micro USB
Standby time	:	At least 30 days
Reading standard	:	ISO 11784/5 FDX-B & ID64
Working frequency	:	134,2 kHz, max. power < 0.5V
Reading distance	:	5 to 13 cm depending on the microchip
Operation temperature	:	- 30 to 50°C
Operation language	:	Universal

The Outpaw PetIdent 160 is a reading device for RFID microchips & RFID ear tags. It is equipped with an OLED screen to display the detected chip number. The Outpaw PetIdent 160 can read all kinds of FDX-B and ID64 RFID tags, with stable, safe and reliable performance.

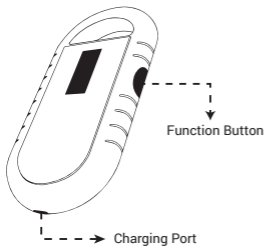Using the Outpaw PetIdent 160

- ◆ Press the function button to turn on the device.
 - ◆ Press the button again to go into scan mode.
 - ◆ Scan the pet or animal with the device. In pets, the microchip is typically located between the shoulder blades. Be aware that in sporadic cases, the chip could also have moved or implanted into a different location.
-
- ◆ A beep tone indicates the positive detection of a microchip. The detected chip number will be displayed on the screen. In case the device is connected to, e.g., a PC using the provided USB cable while reading, the chip number will also be transmitted to your computer. Therefore, e.g., a text file must be opened to record the chip number.
 - ◆ To turn off the reader, then press the button and hold for 3 seconds. Alternatively, the device will turn off itself after 180 seconds.

Charging

- ◆ Connect the Outpaw PetIdent 160 to any USB charger or PC using the provided USB cable.
- ◆ Only use the cable provided in the carton contents for charging the Outpaw PetIdent 160.
- ◆ Do not keep the device on charge for longer than 2 hours!
- ◆ Also, disconnect the device from your computer.

Safety information

- ◆ In case of longer absence, disconnect your Outpaw PetIdent 160 from power.
- ◆ To ensure optimal function of your Outpaw PetIdent 160, store it in a dry place at temperatures between 0°C and 40°C and at a max. relative humidity of 70%.
- ◆ Protect your Outpaw PetIdent 160 against direct sun exposure and water.
- ◆ Safeguard your Outpaw PetIdent 160 from shocks and prevent malfunctions and defects.
- ◆ Do not expose your Outpaw PetIdent 160 to an electrostatic charge.
- ◆ Children should only use the Outpaw PetIdent 160 under parental supervision. Danger of swallowing small parts.
- ◆ The Outpaw PetIdent 160 is not suitable for use by children under the age of 10.
- ◆ Refrain from opening the device's casing. Opening the casing will cause loss of warranty.
- ◆ To ensure a long service life of your Outpaw PetIdent 160, charge it every three months at least.
- ◆ In case of malfunctions or technical defects, please contact your dealer or get in touch with us directly.

Free redemption of old batteries

Your Outpaw PetIdent 160 contains a Lithium battery. Batteries are not to be disposed along with household waste. You are legally required to return old batteries, so it can be ensured that they are disposed properly. You can return old batteries at a communal collection point or at local businesses. Being distributors of batteries, we are also obligated to accept old batteries. However, our redemption obligations are restricted to acceptance of old batteries, which are the same type as new batteries in our portfolio. Therefore, you may return old batteries of this kind in an envelope with sufficient postage.

For technical reasons, we do not recommend replacement of the battery by the consumer/customer. Please get in touch with Outpaw technical service via email at info@outpaw.io for battery replacement.

Meaning of battery symbols

Batteries are designated by the symbol with the crossed-out dustbin (see below). This symbol indicates that batteries may not be thrown into household waste. For batteries containing more than 0.0005 % by mass quicksilver, more than 0.002 % by mass cadmium or more than 0.004 % by mass lead, the chemical designation of the included substances is listed below the dustbin symbol. „Cd“ stands for cadmium, „Pb“ for lead and „Hg“ for quicksilver.

Information on waste electrical and electronic equipment (WEEE)

This appliance is labeled in accordance with European Directive 2012/19/EU concerning used electrical and electronic appliances (waste electrical and electronic equipment - WEEE).

Every customer is legally obligated to dispose of electrical or electronic equipment separately from regular household waste, because they may contain hazardous substances and valuable raw materials.

This equipment is marked with the symbol displayed underneath. It means that it cannot be disposed of as regular household waste.

All equipment can be volunteered for recycling free of charge at a designated and licensed waste facility center. There the equipment will be recycled in an environmentally friendly and resource preserving way.

Customer service information

Thank you for purchasing an Outpaw product.

All our products are protected by our warranty for a period of one year from the initial purchase date. The warranty only protects the product when directly purchased from Outpaw or one of our licensed dealers.

Please note that your warranty claim may be rejected in the following situations:

- ◆ Purchase from an unauthorized dealer or distributor
- ◆ Loss of invoice or receipts (verification of purchase date)
- ◆ Verifiable repair attempt
- ◆ Product damage due to improper packaging or transport
- ◆ Product damage due to a use not included in the user manual.
- ◆ Attempted manipulation or signs of external force

In case of a malfunction or defect of your Outpaw PetIdent 160,

- please return your Outpaw PetIdent 160 to your dealer or
- contact us by e-mail and tell us your problem: info@outpaw.io

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